


ND Hand hygiene training

Areas of use:

- Identification of hygiene problems
- Hand hygiene training for staff and visitors

To be used in:

• Kitchens	• Hospitals
• Laboratories	• Healthcare
• Food production	• Breeding and breeding stables
• Schools	

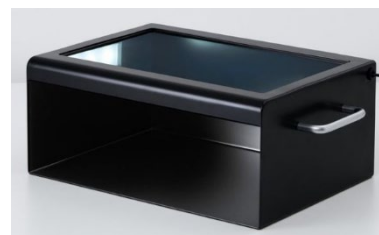

Hand hygiene training:

Poor hygiene can be the cause of many cases of diseases, both in humans but also in relation to spread of infections/ bacteria between production units.

It is estimated that more than 30% of infections in food production could have been avoided by proper hand hygiene.

Method:

1. Add 1-2 pumps of the fluorescent lotion/ hand sanitizer and use it as it was a normal thorough cleaning of hands.
2. Place your hands in the cabinet. The entire surface of both hands should shine evenly. Repeat if necessary, no 1 until the entire surface is uniformly lit.
3. Wash your hands thoroughly. Wipe them and check again in the cabinet. The areas that still lit are not washed sufficiently.
4. Repeat as needed and look for improvements.

Technical data:

• Name	ND Hand hygiene training	• Height	150 mm
• Material	Aluminum	• Width	357 mm
• Color	Black	• Length	251 mm
• Weight	1 kg	• Light type	LED